

# Aptima® Urine Collection Kit


## Collection procedure guide

### Collection for male and female urine specimens

Patient should not have urinated for at least 1 hour prior to specimen collection.



Direct patient to provide **first-catch** urine (approximately 20 to 30 mL of initial urine stream) into urine collection cup free of any preservatives. Collection of larger volumes of urine may result in specimen dilution that may reduce test sensitivity. Female patients should not cleanse labial area prior to providing specimen.

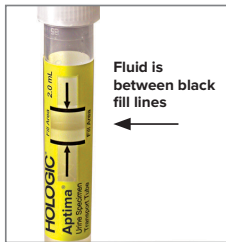

Remove cap from urine specimen transport tube and transfer 2 mL of urine into urine specimen transport tube using the disposable pipette provided. The correct volume of urine has been added when the fluid level is between the black fill lines on urine specimen transport tube label.



Re-cap urine specimen transport tube tightly. This is now known as the “processed urine specimen.”

#### Urine specimen collection guide for:

- *Chlamydia trachomatis* (CT)
- *Neisseria gonorrhoeae* (NG)



### Specimen transport and storage




- Urine samples still in primary collection container must be transported to the lab between 2°C to 30°C. Transfer urine sample into Aptima urine specimen transport tube within 24 hours of collection for Aptima Combo 2® assay for CT/NG assay.
- Transport and store processed urine specimens in the Aptima urine specimen transport tube between 2°C to 30°C until tested.



**Overfilling or underfilling will require a new sample collection tube to obtain a valid result.**

Transfer urine into collection tube.

Fill between black lines only.

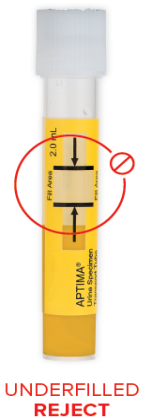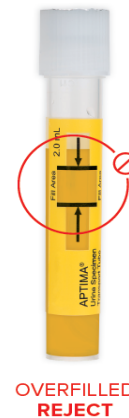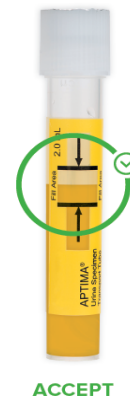

Hologic provides this collection procedure guide as a general informational tool only; it is not an affirmative instruction or guarantee of performance. It is the sole responsibility of the clinician/laboratory to read and understand the appropriate package insert and comply with applicable local, state and federal rules and regulations.

[hologic.com](http://hologic.com) | [diagnostic.solutions@hologic.com](mailto:diagnostic.solutions@hologic.com) | 888.484.4747

DS-01697-001 Rev. 002 © 2017 Hologic, Inc. All rights reserved. Hologic, The Science of Sure, Aptima Combo 2, Aptima and associated logos are trademarks and/or registered trademarks of Hologic, Inc. and/or its subsidiaries in the United States and/or other countries. This information is intended for medical professionals in the U.S. and is not intended as a product solicitation or promotion where such activities are prohibited. Because Hologic materials are distributed through websites, eBroadcasts and tradeshows, it is not always possible to control where such materials appear. For specific information on what products are available for sale in a particular country, please contact your local Hologic representative or write to [diagnostic.solutions@hologic.com](mailto:diagnostic.solutions@hologic.com).